BAUERFEIND AG

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 english

Dear Customer,

Thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read and closely observe the specifications in these instructions for use and the instructions given by the professional. If you have any questions, please contact your physician or medical retailer.

Intended purpose

ViscoBalance is a medical device for the feet. It is a viscoelastic heel cushion to compensate for different leg lengths of 3, 5 and 10 mm. With the heel wedge, the product also offers relief of the Achilles tendon.

Indications

- Leg length differences
- Achillodynia (tendinitis, bursitis subachillea)

Risks of using this product

Important information

• **First-time application / fitting of the ViscoBalance may only be carried out by a professional¹, otherwise product liability cannot be assumed.**

- ViscoBalance is a prescribed product that should be applied under a physician's guidance. ViscoBalance should only be applied in accordance with the information contained in these

instructions for use and for the listed indications.

- Product liability cannot be assumed in the event of inappropriate use.
- Do not alter the product in any way. Failure to comply may adversely affect the product's performance, thus canceling any product liability.
- If there is not enough space inside the shoe for the foot and the selected heel cushion, or if heel cushions are worn in shoes of an incorrect length and width (or shoes with inadequate lateral guidance in the case of open shoe types), problems relating to poor fit may arise (e.g. a feeling of constriction or pressure, rubbing). If the feet are positioned on the edges of the foot orthoses, blisters or calluses may form on the soles of the feet. This may also occur if the material develops holes or breaks due to mechanical overloading. You must stop wearing heel cushions if they are damaged in this way!
- If you notice any changes or an increase in symptoms while wearing the product, stop any further use and contact your physician.
- Please make sure to consult your physician before using the ViscoBalance during exercise.

Contraindications

Side effects involving the body as a whole have not been reported to date. Correct usage / donning of the product is assumed.

No clinically significant hypersensitivity reactions have been reported to date. If you have any of the following conditions, such aids should only be donned and worn after consultation with your physician:

Skin disorders/injuries to the relevant part of the body, particularly if inflammation, arterial occlusive vascular disease, or disorders of the lymph system are present. Likewise, any raised scars with swelling, redness, and excessive heat build-up. People with diabetes or rheumatoid foot deformities must seek medical advice before using this product to ensure optimum treatment.

Application instructions

Put the ViscoBalance heel cushion into the shoe so that it fits snugly in the heel area.

Cleaning instructions

We recommend washing ViscoBalance with lukewarm water.

Notes on reuse

This product is intended to be fitted to one patient only.

Part of the body this product is used for

The foot orthoses must be worn in suitable footwear and with socks or stockings on your feet.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty.

The product must be cleaned before submitting it in case of warranty claims. The warranty may be limited or excluded if the instructions on how to handle and care for the ViscoBalance have not been observed. Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the professional were not observed
- The product was modified arbitrarily

Disclaimer

Do not diagnose yourself or decide on which medication to take unless you are a professional¹. Before first using our medical device, please seek the advice of a physician or professional¹ as this is the only way to assess the effect of our product on your body and determine the potential risks of using this product due to your personal constitution. Follow the

advice of this professional as well as all instructions in this document or its online version, including any excerpts (as well as texts, images, graphics etc.).

If you still have doubts after your consultation with the professional, please contact your physician or distributor, or contact us directly.

Duty to report

Due to regional legal regulations, you are obliged to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

Silicone (SI)

 – Medical device

 – DataMatrix Unique Device Identifier UDI

Date of information: 2025-03

¹ A professional is any person who is authorized according to the state regulations applicable for fitting and instructing in the use of orthopedic foot orthoses.

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Bauerfeind Middle East FZ LLC
Bauerfeind USA, Inc.