BAUERFEIND AG
 Triebeser Straße 16
 07937 Zeulenroda-Triebes
 Germany
 P +49 (0) 36628 66-40 00
 F +49 (0) 36628 66-44 99
 E info@bauerfeind.com
BAUERFEIND.COM

ValguLoc® II

Hallux-valgus-Orthese
 Hallux valgus orthosis
 Orthèse hallux valgus
 Tutore per alluce valgo
 Ortesis hallux valgus

1

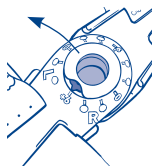
2

3

4

5

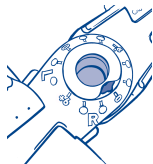
6

7

8

9

de	deutsch	3	hr	hrvatski	64
en	english	8	sr	srpski	68
fr	français	12	sl	slovenski	72
nl	nederlands	16	ro	romanian	76
it	italiano	21	tr	türkçe	81
es	español	25	ru	русский	85
pt	português	30	et	eesti	90
sv	svenska	35	lv	latviešu	94
fi	suomi	39	lt	lietuvių	98
da	dansk	43	he	עברית	102
pl	polski	47	ko	한국어	105
cs	česky	51	ar	عربي	111
sk	slovensky	55	zh	中文	113
hu	magyar	60				

de deutsch

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Dear Customer,

thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read and observe these **instructions for use** carefully. If you have any questions, please contact your doctor or medical retailer.

Intended purpose

ValguLoc II is a medical product. It is an orthosis¹ for the correction of the big toe at rest and during movement.

⚠ The ValguLoc II must be individually adjusted for the patient by a trained specialist³. This is the only way to guarantee its full effect and optimal wearing comfort.

Indications

- Post-operative protection / rehabilitation after hallux valgus surgery
- Non-surgical functional treatment of big toe misalignment with mobilization

Risks of using this product

⚠ Caution²

Please closely observe the specifications in these instructions for use and the instructions given by the specialist.

- The ValguLoc II orthosis is a prescribed product that must be used under a physician's guidance. The ValguLoc II should only be worn in accordance with these instructions for use and for the listed areas of application.
- Product liability cannot be assumed in the event of incorrect use of the product.
- No improper modifications may be made to the product. Failure to comply with this requirement

may adversely affect product performance, thereby excluding any product liability.

- Should you notice increased symptoms or any unusual changes, stop using the product and please contact your physician without delay.
- Use in combination with other products, e.g. when wearing compression stockings as part of compression therapy, must be discussed beforehand with your physician.
- Do not allow the product to come into contact with ointments, lotions, or substances containing grease or acids.
- Side effects involving the body as a whole have not been reported to date. Correct usage / fitting is assumed. Any aids applied externally to the body – supports and orthoses – can, if tightened excessively, lead to local pressure signs and can disrupt wound healing after surgery, or, in rare cases, constrict the underlying blood vessels or nerves. If this happens, loosen the straps on the ValguLoc II and have the size of your ValguLoc II checked if necessary.

If you would like to use your Bauerfeind product due to acute symptoms / injuries, make sure to obtain and follow the advice of a medical professional before first using the product. Restrictions may exist given the specific indication which the specialist retailer can clarify. In particular, the ability to operate vehicles, other means of transport or machinery may be limited. In case of doubt, we recommend temporarily refraining from the activities mentioned above.

Contraindications

No clinically significant hypersensitive reactions have been reported to date. If you have any of the following conditions, such aids should only be fitted and

worn after consultation with your physician:

1. Wounds in the area in contact with the orthosis
2. Skin disorders / injuries to the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness, and excessive heat build-up
3. Impaired sensation and movement disorders of the leg / feet, e.g. "diabetes mellitus"
4. Impaired lymph drainage – including soft tissue swellings of uncertain origin located away from the fitted aid.
5. Gout of the big toe (podagra)

Application instructions

Fitting the ValguLoc II orthosis

The dynamic orthosis for the big toe can be worn on the right or left foot. When fitting it, make sure that the marking for the side to be treated is visible at the top of the scale. When fitted to the right foot, the R (for right) in light gray lettering should be visible. When fitted to the left foot, the L (for left) in red lettering should be visible. If the right and left are mixed up, the orthosis may slip, resulting in adverse effects on the metatarsophalangeal joint and loss of therapeutic effect.

The ValguLoc II is supplied from the factory with a joint stop used after surgery. To release the stop, please proceed as follows:

1. Fold back the internal cushioning over the joint to reveal the inner locking ring with two »lugs« on opposite sides. **1**
2. Remove this locking ring as this is not required for conservative treatment. It is not possible to walk with the orthosis when the locking ring is in place, as the joint remains immobile **2**. The locking ring is easier to remove if you remove the outer locking ring first **3** and then push the inner locking ring inwards from the outside.

3. Now position the orthosis on the inner side of the foot in question. Open the straps that point upwards and pull these out of the guide slit. Make sure that the toe shell is on the side of the big toe and the foot shell is on the side of the foot. The joint of the orthosis must be exactly centered on the inside over the middle of the metatarsophalangeal joint (ball). **3**
4. Place the small strap around the big toe, starting from the bottom. Position the large strap underneath the foot. Close both straps by placing the toe strap around the big toe starting from the bottom, pull it through the guide slit, and fasten it lightly. Place the foot strap around the midfoot, again starting from the bottom, pull it through the guide slit, and fasten it lightly. Then under the end of the foot strap again and adjust it until the orthosis fits firmly against the foot, but without any uncomfortable pressure. Then do the same with the toe strap. **4** As supplied, the toe strap is long enough to fit round a bandaged toe after hallux valgus surgery. When used without a bandage, the strap can be shortened during the adjustment (cut the Velcro fastening with scissors).

The orthosis is correctly positioned on the foot if movement of the metatarsophalangeal joint is largely unimpaired. It can then be worn in a shoe. To avoid unnecessary pressure points, you should only wear the orthosis in a wide, comfortable shoe (e.g. a sport shoe).

If the orthosis is used for immobilization after surgery, any movement in the metatarsophalangeal joint must be prevented (lock the joint of the orthosis). When used postoperatively, the orthosis may only be worn in a strap shoe with a reinforced sole.

Adjusting and locking the correction angle

The varus-valgus angle of the orthosis can be adjusted in steps from the outside by means of an eccentric adjuster (in approx. 2° increments from 10° valgus to 2° varus) using an eccentric adjuster (setting as supplied from the factory +6°). For an orthosis to be worn on the right, the angle must be adjusted in direction R (gray color scale) and for an orthosis to be worn on the left it must be adjusted in direction L (red color scale). Please proceed as follows:

- Remove the outer locking ring on the orthosis using the adjuster chip supplied 5. Press the flattened side of the adjuster chip against the side of the outer locking ring from the bottom in an upward direction to release the ring from its anchorage and reveal the adjuster ring.
- Remove the locking ring for the joint stop as described in points 1 and 2 and fit the orthosis to the foot in question as described in points 3 and 4. The correction angle can now be adjusted on the fitted orthosis 6. If the foot is painful or postoperative, the correction angle should not be adjusted while the orthosis is fitted to the foot.
- To adjust the angle, insert the adjuster chip supplied (or with a suitable coin) into two grooves on opposite sides and turn them until the pointer shows the desired angle 7.
- To lock the adjusted angle, press the outer locking ring back into the adjuster ring 8 by placing the notch in the outer locking ring over the pointer in the adjuster ring and pressing the outer locking ring firmly until it clicks into place 9.

Cleaning instructions

The ValguLoc II can be washed separately by hand with a mild detergent. To prevent damage and to

preserve their long-term adhesive strength, we recommend closing the Velcro fastenings before washing. Do not dry in direct heat (e.g. heater, sunlight, etc.).

If you have any complaints, please contact your medical retailer only. Please note that only clean goods can be dealt with.

Part of the body this product is used for

According to the indication (big toe)

Maintenance instructions

If handled correctly, the product requires virtually no maintenance.

Assembly and fitting instructions

The ValguLoc II is supplied ready for use.

Technical specifications / parameters

The ValguLoc II is a hallux valgus orthosis with a 3D joint and two attached straps. The straps are removable and available as spare parts.

Notes on reuse

This product is intended to be fitted to one patient only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. The warranty may be limited or excluded if the instructions on how to handle and care for the ValguLoc II have not been observed. Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist³ were not observed.
- The product was modified arbitrarily

Disclaimer

Do not diagnose yourself or decide on which medication to take unless you are a medical specialist.

Before first using our medical product, please seek the advice of a physician or trained specialist, as this is the only way to assess the effect of our product on your body and determine the potential risks of using this product due to your personal constitution. Follow the advice of this specialist as well as all instructions in this document or its online version, including any excerpts (as well as texts, images, graphics etc.).

If you still have doubts after your consultation with the specialist, please contact your physician or retailer, or contact us directly.

Duty to report

Due to regional legal regulations, you are obliged to immediately report any serious incident involving the use of this medical product to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

Polyamide (PA), Glass fiber (GF), Ethylene Vinyl Acetate (EVA), Polyurethane (PUR)

 – Medical Device

 – DataMatrix Unique Device Identifier UDI

Version: 2023-02

⚠ Information for specialists

Joint locking after surgery or other indication

- To lock any movement of the joint of the orthosis, the locking ring

must be clicked in. As supplied, the orthosis is locked in 0° of flexion.

- Release the joint stop as described in points 1 and 2.
- Before you can lock the joint in the desired position, undo the straps and hold the orthosis in your hand. Place the orthosis on the side of the metatarsophalangeal joint to be treated and take note of this position.
- Then fold back the front cushioning on the inside. You will now see a toothed grating into which you can click the locking ring 2. Adjust the desired angle of flexion and click the locking ring into the toothed grating. Make sure that the two lugs on the locking ring fit into the locking notches. The hallux valgus joint is locked.
- Fold the cushioning back into its previous position and put the orthosis back on the patient.
- The orthosis is now locked and ready to use.
- To release it after a few days, remove the locking ring again as described in points 1 and 2 and also check that the varus-valgus angle is correct.
- Finally, pull the white separating paper from the inside of the front cushioning and press the cushioning against the toe shell. The cushioning is now attached to the toe shell. This prevents the toe cushioning from slipping during movement.

Splay-foot pad

If necessary, the orthosis can be used in combination with a splay-foot pad. In this case, an adhesive pad needs to be selected ensure secure positioning inside the shoe.

- Orthosis = orthopedic aid used to stabilize, relieve, immobilize, control, or correct the limbs or the torso
- Note on the risk of personal injury (risk of injury, harm to health, and accidents) or damage to property (damage to the product)

3 A specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.

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Bauerfeind Middle East FZ LLC
Bauerfeind USA, Inc.